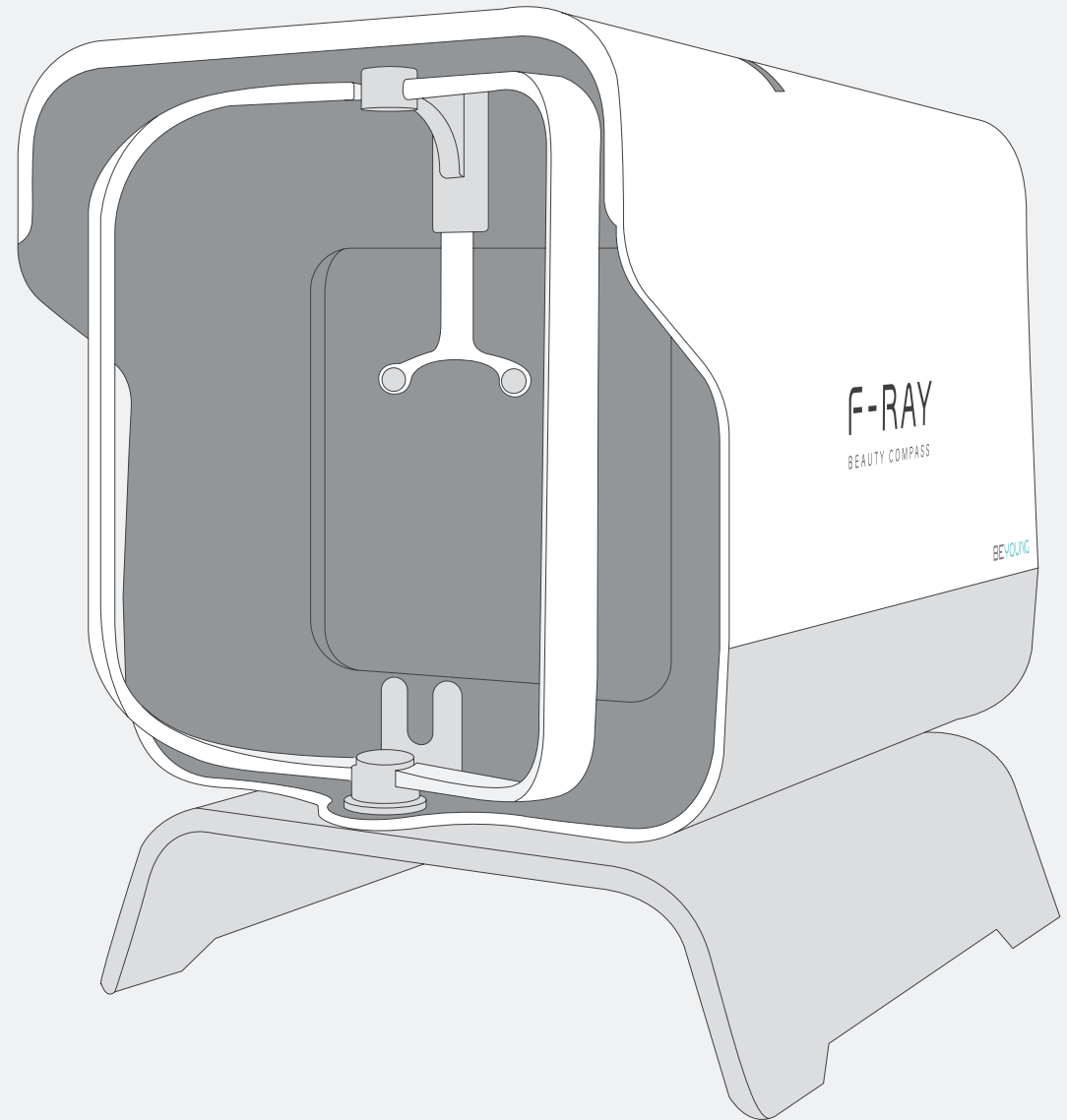
F-RAY

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Operating Manual

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Thank you for choosing to buy F-RAY.

Before using this device, read all instructions and cautionary markings in this manual.

Service, support and repair charges may apply if damage is caused by not following the instructions in this manual.

Read and save these instructions.

Contents

Important Safeguards	20
Product Description	21
Getting Started	22
Preparing to Shoot	23
Face Positioning	25
Adjusting Preferences and Shooting	26
Checking on Images	29
Troubleshooting	32
Care and Maintenance	32
Technical Specifications	33
System Requirements	33

Important Safeguards

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- ▲ Place the product on a flat surface.

Otherwise the product may fall down, causing damage to the product or to persons.

- ▲ Do not use the product in the following places since it may cause electrocution or fire.

- Near gas stoves or near open flames
- Near humidifiers or areas with high humidity
- Areas exposed to direct sunlight or where the temperature may rise significantly
- In/around flammable gases or organic solvents

- ▲ Always unplug every cable from the product before moving or maintenance.

To unplug cables, always hold the plug to remove the cable, and do not pull the cord.

- ▲ Do not use if there is damage to the power cord or USB cable.

Make sure the power cord and USB cable aren't bent or twisted while using or keeping them. Doing so may damage the product.

CAUTION

Caution indicates a potentially hazardous situation which, if not avoided, could result in minor/moderate injury or damage to property.

- ⓘ Remove all packaging materials before using.

Check that all contents are included in the package and not damaged during shipping. In case of product damage or apparent defect, please contact the provider.

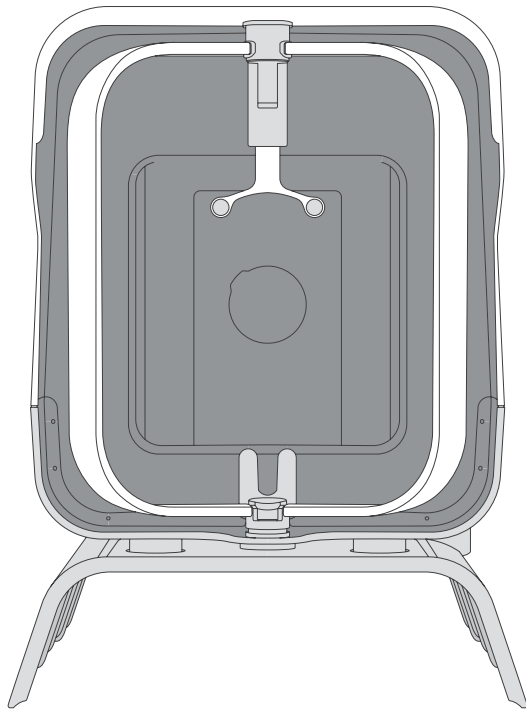
- ⓘ Do not apply excessive force to the product.

Excessive movement and handling may cause a product damage.


Product Description

Contents of the package


The product is delivered as a complete unit.


main device × 1

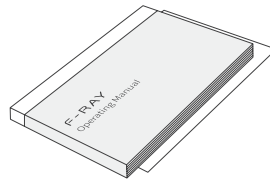
operating manual, warranty card × 1

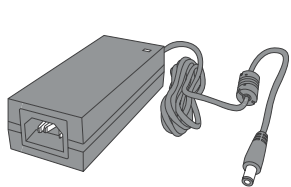AC adapter × 1


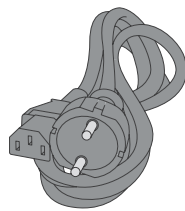
power cord × 1

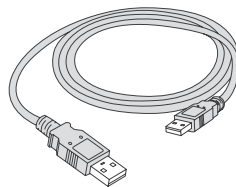USB cable × 1

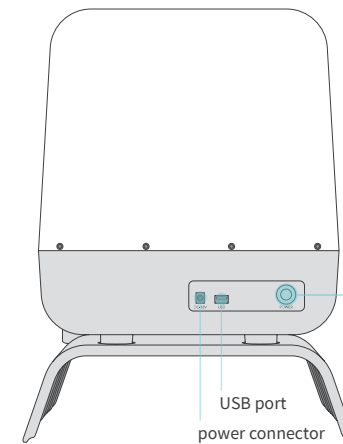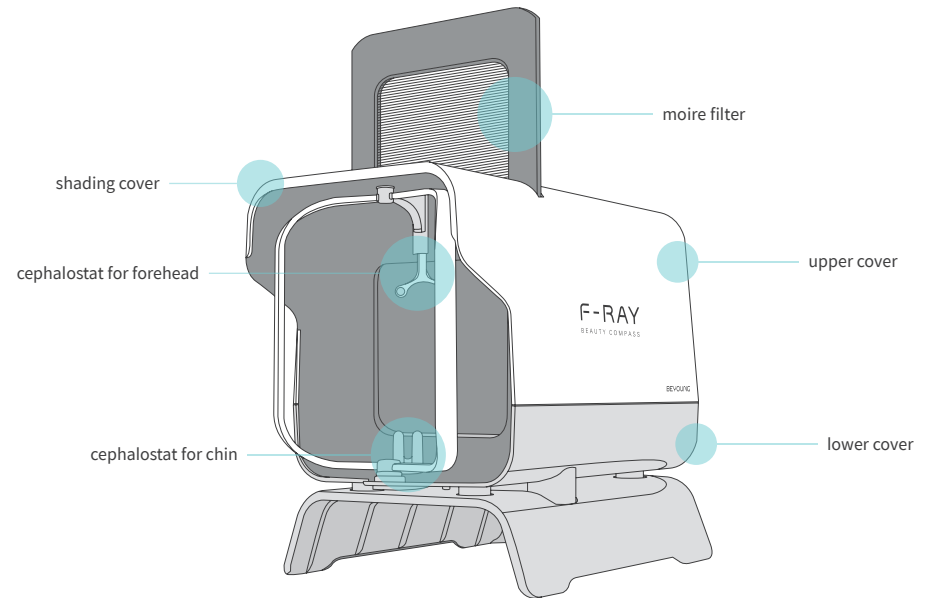Names of parts

Getting Started

1 Download F-RAY application setup file on PC

Go to below address to download F-RAY PC application setup file.
<http://beyoung.co.kr/en/download.html>


* If there is any need to change camera settings, please ask your dealer.

2 Install F-RAY application on PC

Double-click the icon of downloaded setup file to install.
Follow the directions on the pop-up windows to install the software.


3 Connect F-RAY to power supply

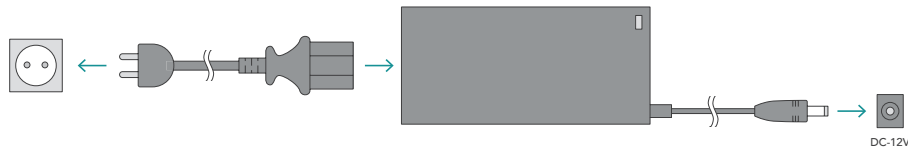Plug the power cord into the AC adapter, and connect the DC connector end of the AC adapter into the F-RAY power connector. Plug power cord directly into a functioning wall outlet.

4 Connect F-RAY to PC

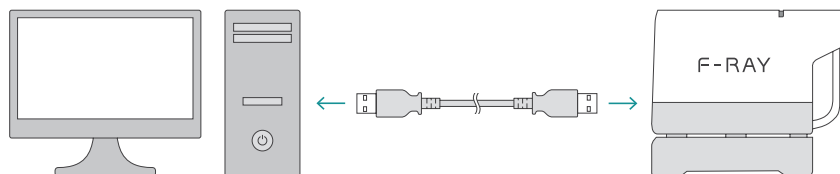Plug one end of the USB cable into a USB port of PC, and plug another end into the F-RAY USB port.


5 Turn F-RAY on

Push the power button on F-RAY to turn on. Now you're ready to use F-RAY.

Preparing to Shoot

1 Launch F-RAY application

Double-click the shortcut icon of F-RAY application on your desktop.


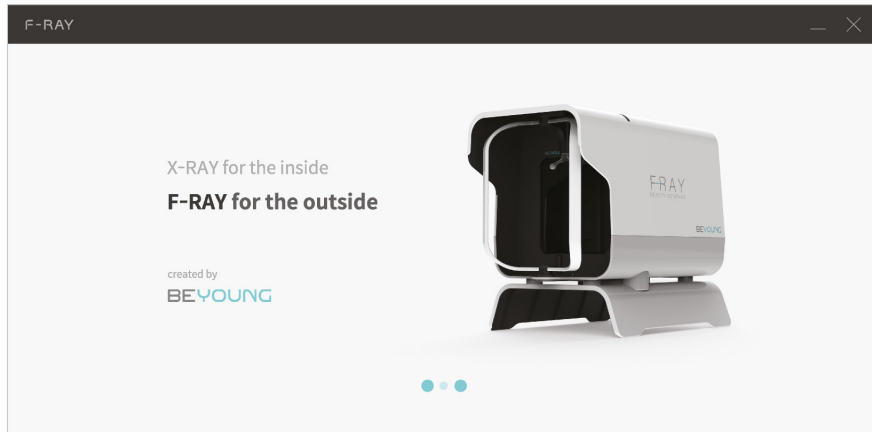
* To proceed without connection problems, turn on the device first and then launch the application.

* If you get “Connection Failed” message, go to page 15 for troubleshooting instructions.

2 Select save location for images

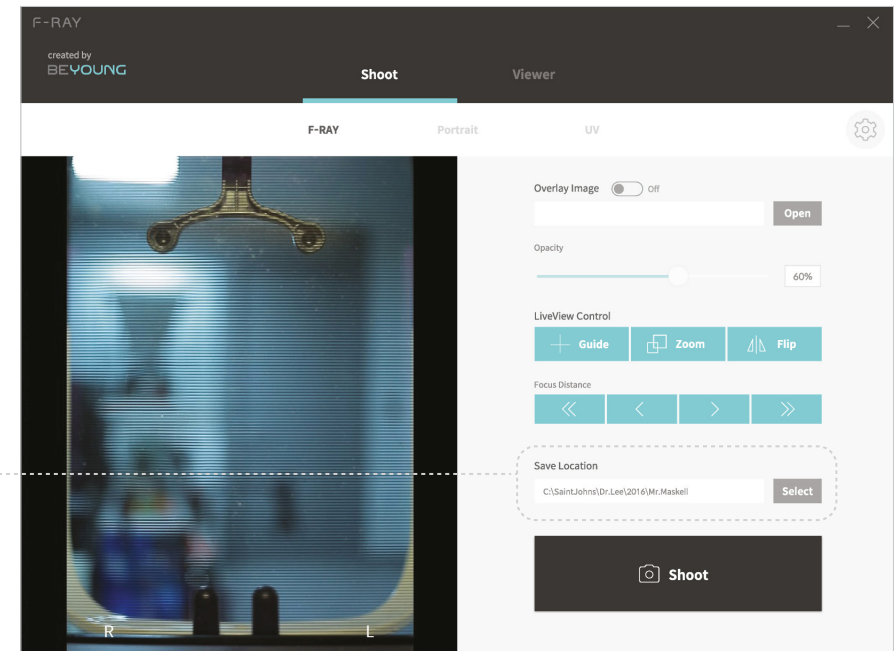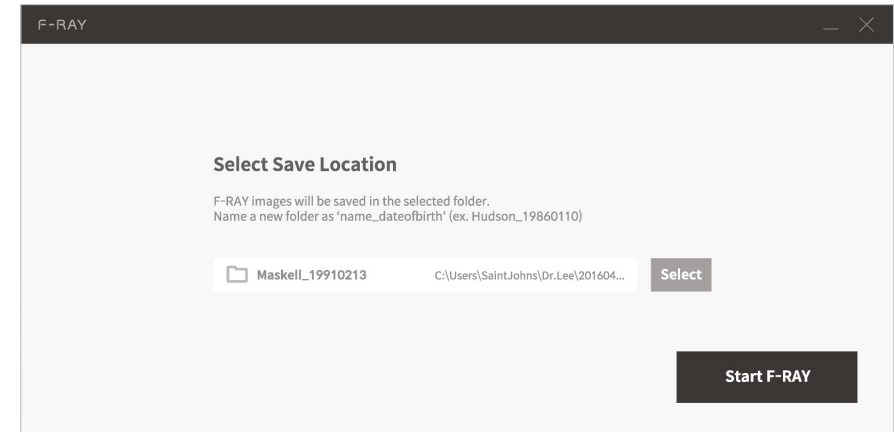
Select a folder where you want to save the images taken by F-RAY.

You can change save location anytime on shoot panel.

Preparing to Shoot

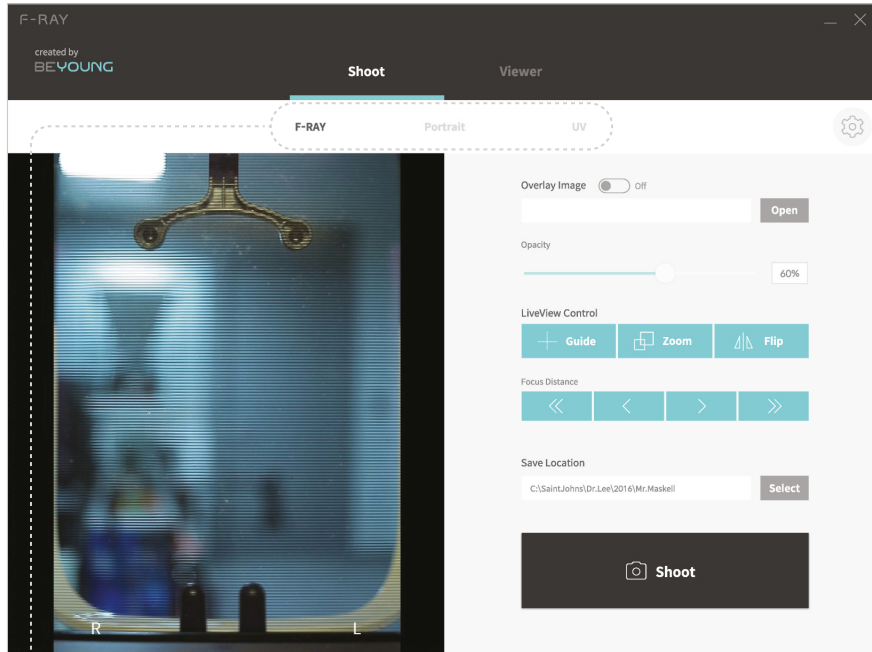3 Select shooting mode


- F-RAY mode (for contour lines)
Images captured under F-RAY mode have contour lines which are helpful for understanding facial shapes and effectiveness of procedures intuitively and accurately.
- Portrait mode (for normal pictures)
Images are captured without fisheye effect under portrait mode and are almost identical to what human eyes see.
- UV mode (for facial spots and blemishes)
Images captured under UV mode expose freckles, facial spots, blemishes, and pores which are usually unseen by human eyes.

* Be aware that there can be a move of the filter when changing between different shooting modes.

4 Check on LiveView

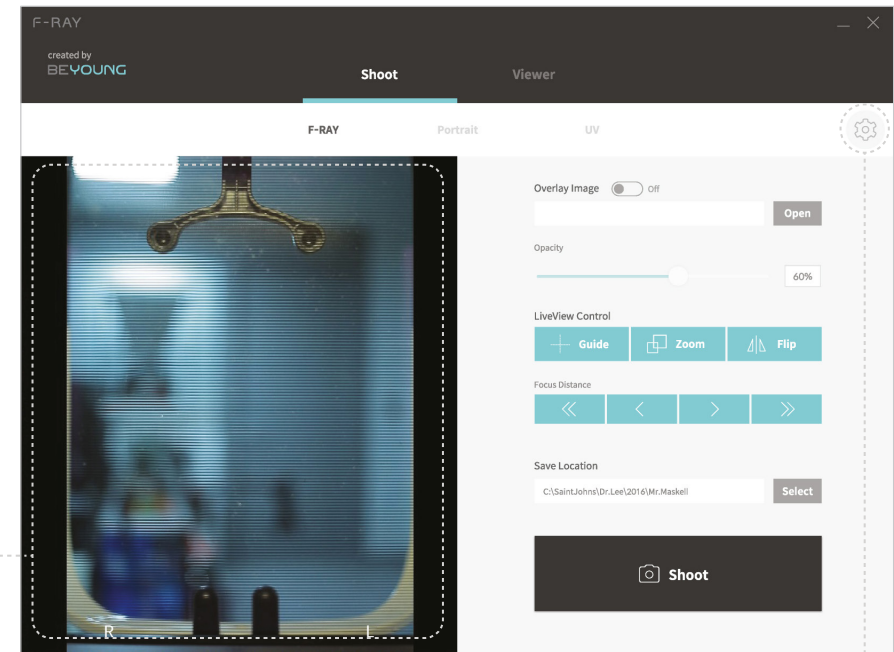Move a hand in front of the product and check if you can see it on LiveView to make sure it is working properly.


* This button is for changing advanced camera settings which shouldn't be changed by user. If you need any further assistance, please contact your dealer.

- ! Do not click anything in the application until the filter stops moving.
- ! If an odd noise occurs when using the product, remove it from power supply immediately and contact BEYOUNG customer services.

Face Positioning

1 Position patient's face

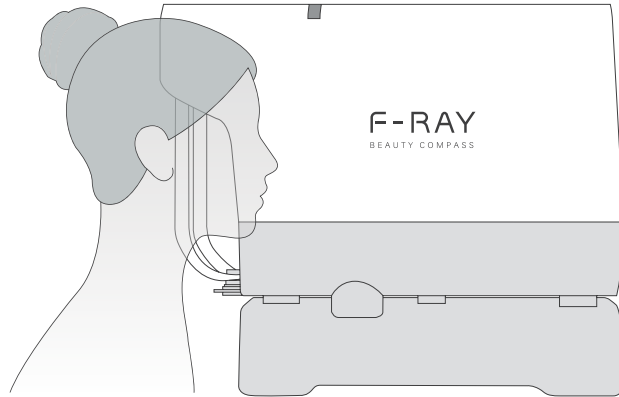
Position patient's face on F-RAY.


! Be careful not to hit patient's head on the product.

2 Balance face position

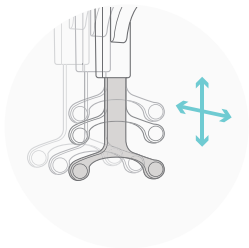
Adjust cephalostats to balance patient's face position. You can check the position and symmetry on LiveView in real-time.

cephalostat
(for forehead)

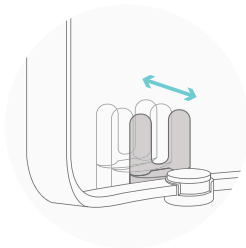
cephalostat
(for chin)

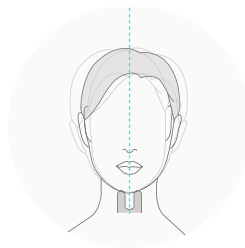symmetry check

3 Adjust face angle

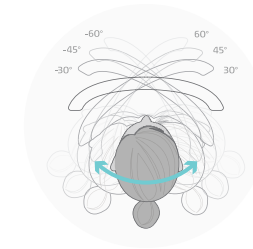Images can be captured in seven different angles by using angle adjuster.
(front, left/right 30°, 45°, 60°)

angle adjuster

60°

45°

30°

front

30°

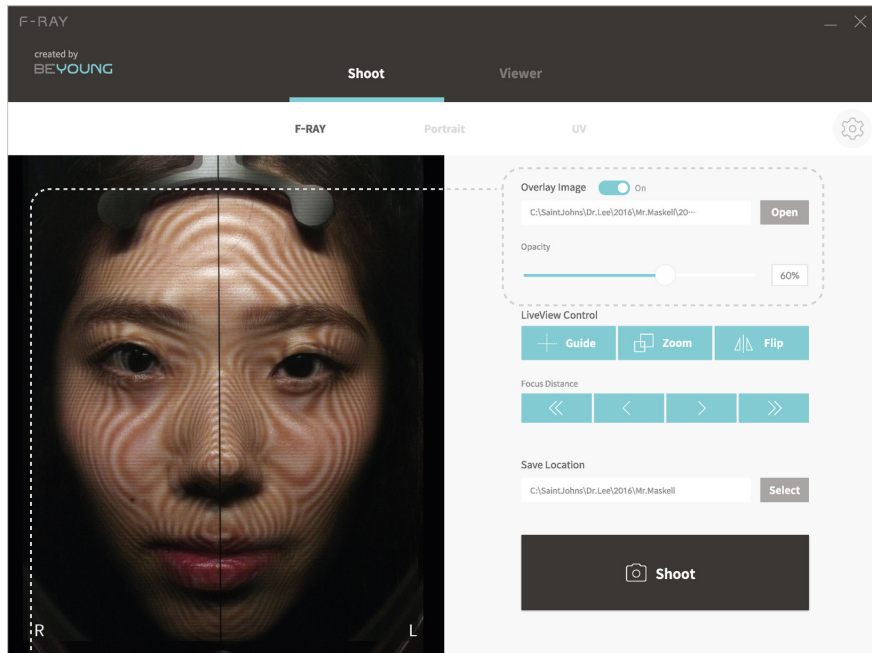
45°

60°

Adjusting Preferences and Shooting

1 Overlay previously captured image


If there are any previously captured images, click Open and select the one that you want to use as a reference for comparison. This feature is helpful for comparing before/after images precisely.

- **Overlay Image**
You can show or hide the image overlay by toggling the switch.
- **Opacity**
You can change the opacity of the image overlay by moving the slider's handle.


2 Adjust LiveView screen

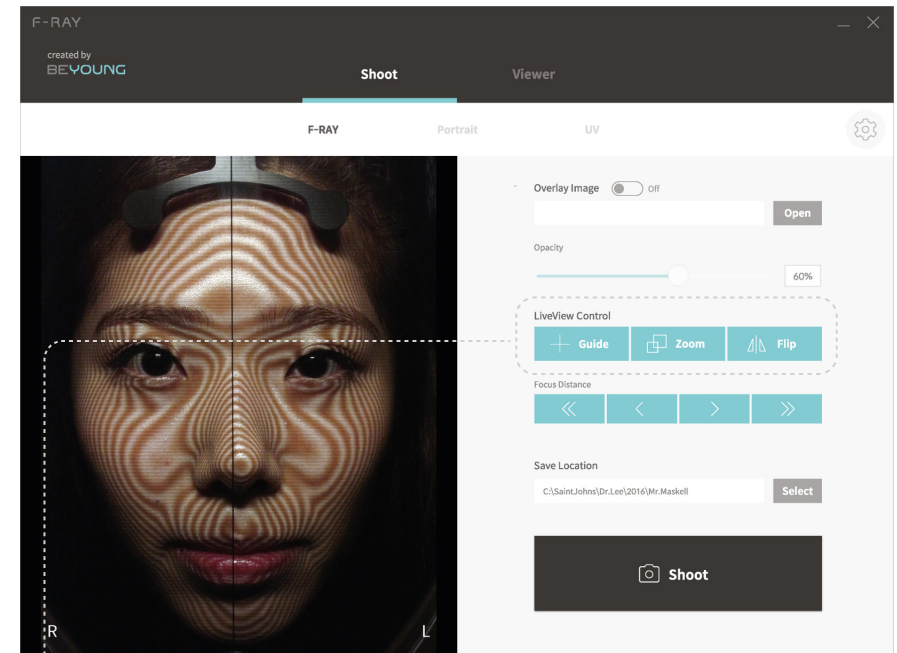You can adjust LiveView screen by using LiveView Control. Currently F-RAY provides three LiveView Control features.


- **Guide**
Show crossed guides on LiveView.
- **Zoom**
Zoom LiveView screen by 150%.
- **Flip**
Flip LiveView screen horizontally.

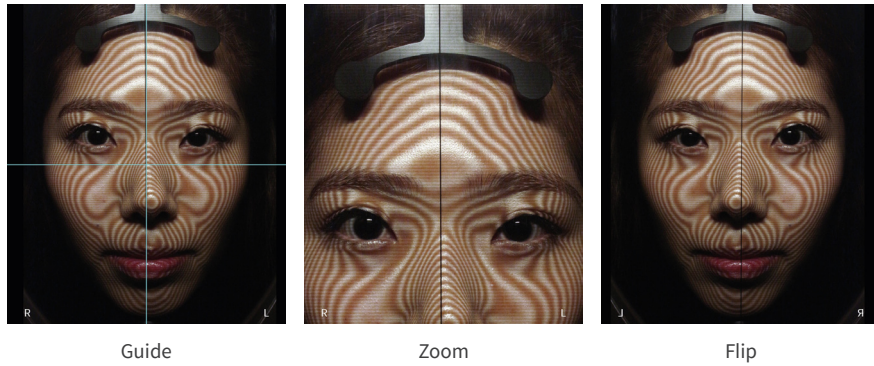* When you capture an image while 'Flip' feature is turned on, a horizontally flipped image will be saved. Be aware that the saved image is a mirror image

3 Adjust focal length

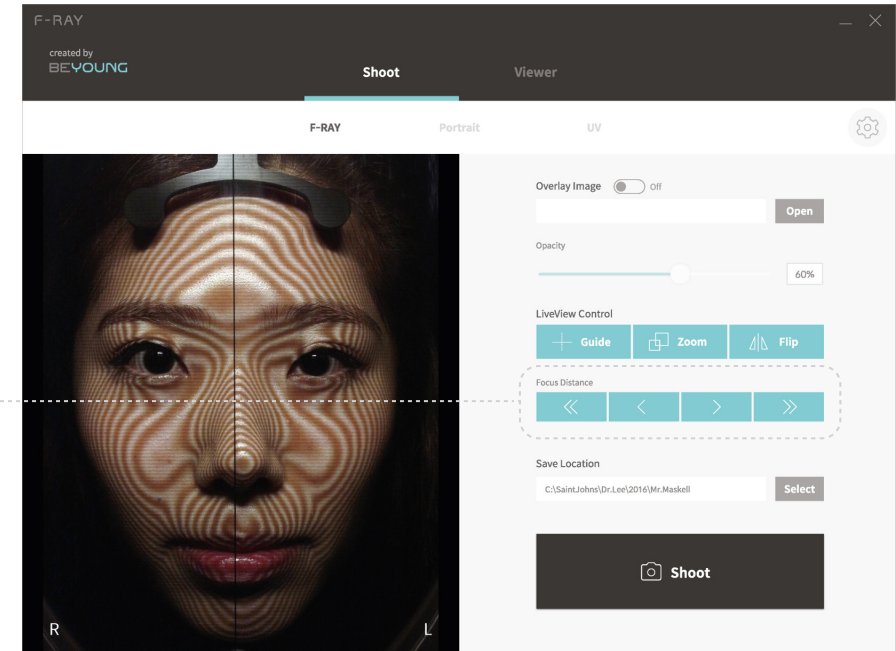Adjust focal length to focus camera. You can maximise the image quality by adjusting focal length while LiveView control's 'Zoom' feature is turned on.

4 Capture image

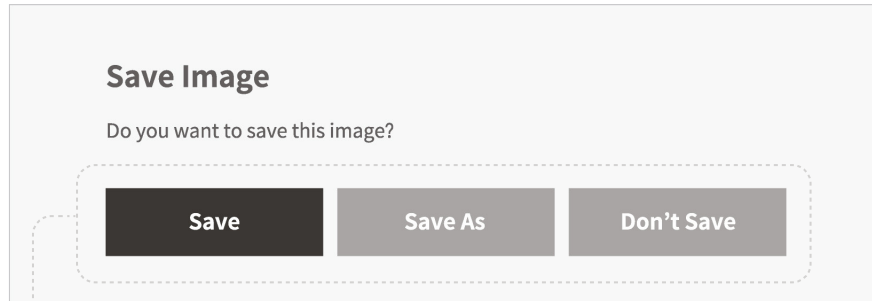
Click 'Shoot' to capture image.

* Due to the nature of the product, while capturing images under F-RAY mode, the position of the filter's bottom line may differ in different images. Please keep in mind that this is not a malfunction.

Adjusting Preferences and Shooting

5 Save image

Save the captured image.


- **Save**
Save the image without changing the name of the folder where it will be saved. The image will be saved in an automatically created folder with current date as its name.
- **Save As**
Set the name of the folder manually.
- **Don't Save**
Discard captured image.


Checking on Images

1 Open viewer panel

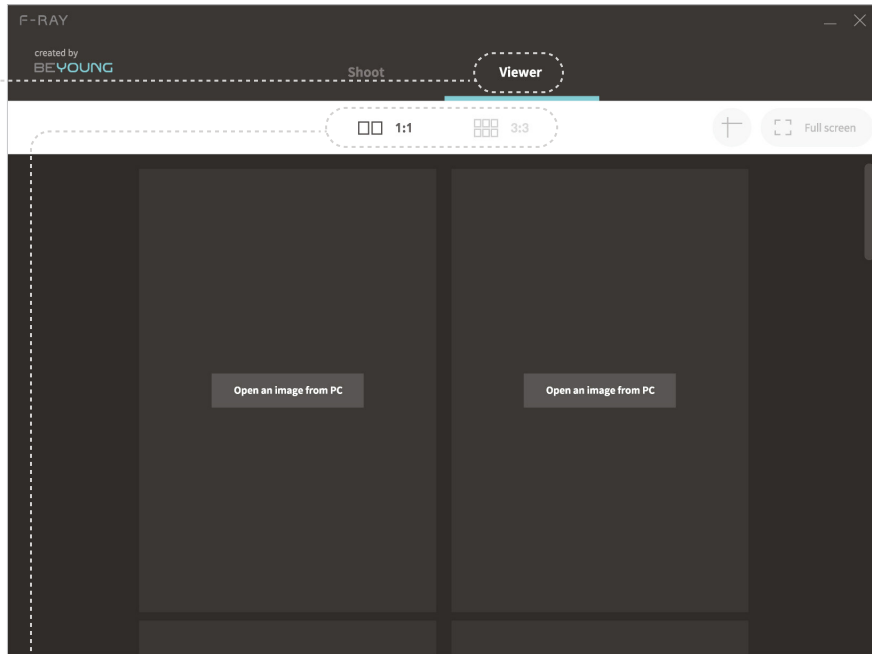
Click 'Viewer' tab to check and compare images.

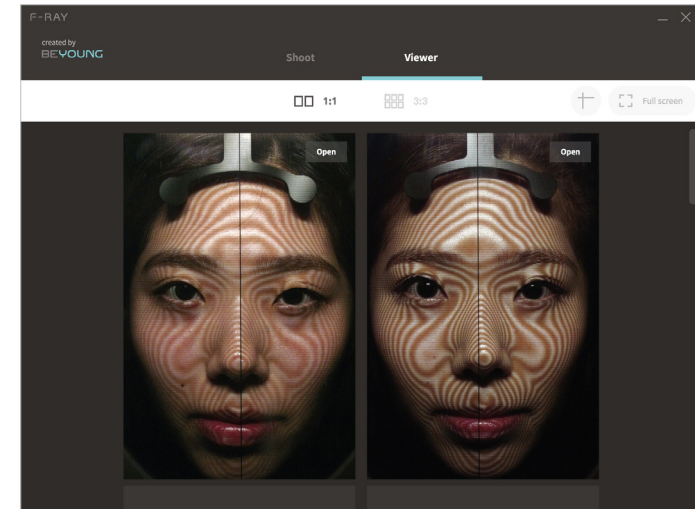- 1:1 mode
Suitable for checking on images or comparing 1:1 before/after images.
- 3:3 mode
Suitable for comparing before/after images captured in various angles.

1:1 mode

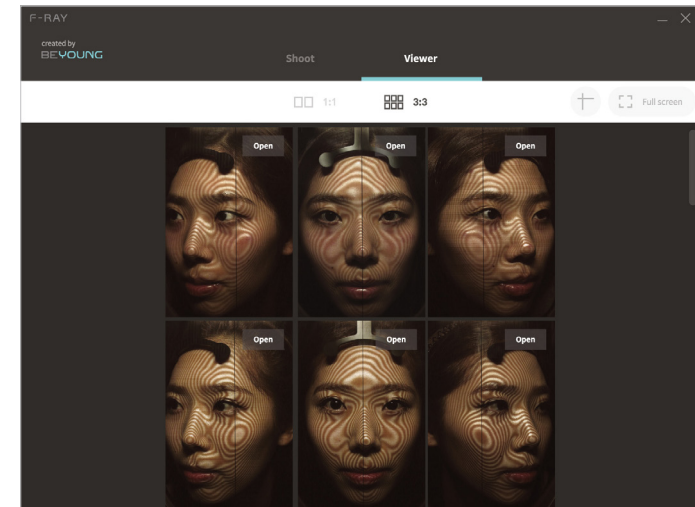
3:3 mode

Checking on Images

2 Load image

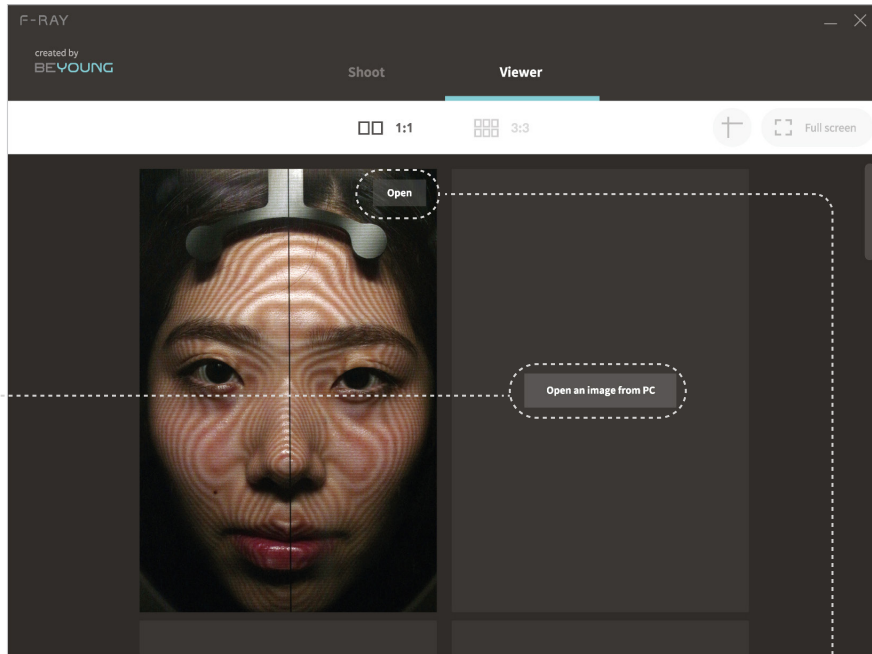
Click 'Open an image from PC' to load an image.


* Loaded image can be changed by clicking 'Open' and selecting another image.

3 View in full screen

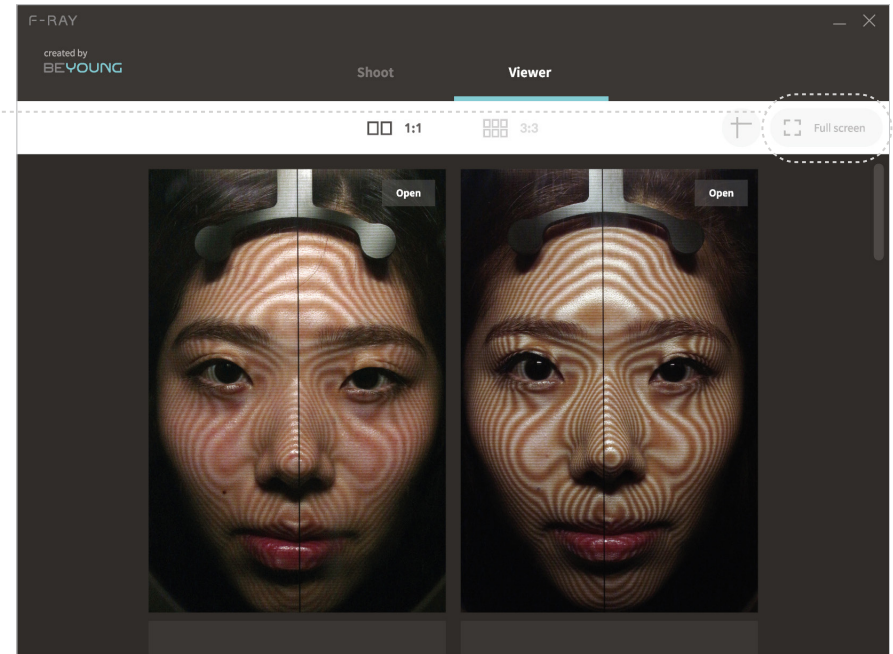Click 'Full screen' to view images in full screen.

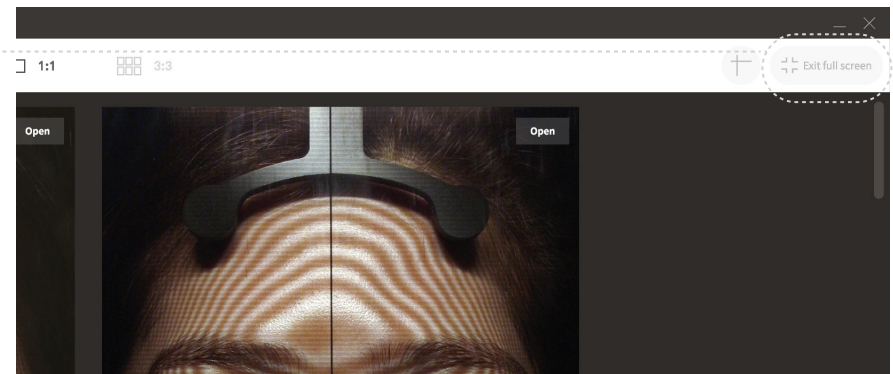
Exit full screen by clicking 'Exit full screen'.

4 Show guides

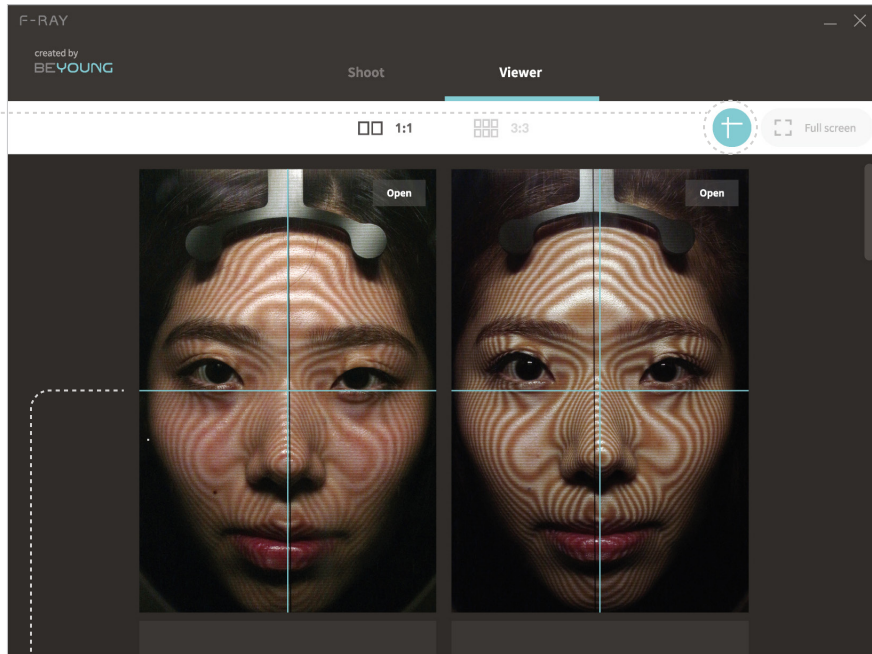
Click guides button to show guides on images.

- * You can move a guide to different position by dragging it.
- * When you move a guide to different position, every same guide on other images will also be moved to the same position. Use this feature for comparing before/after images

5 Zoom into images

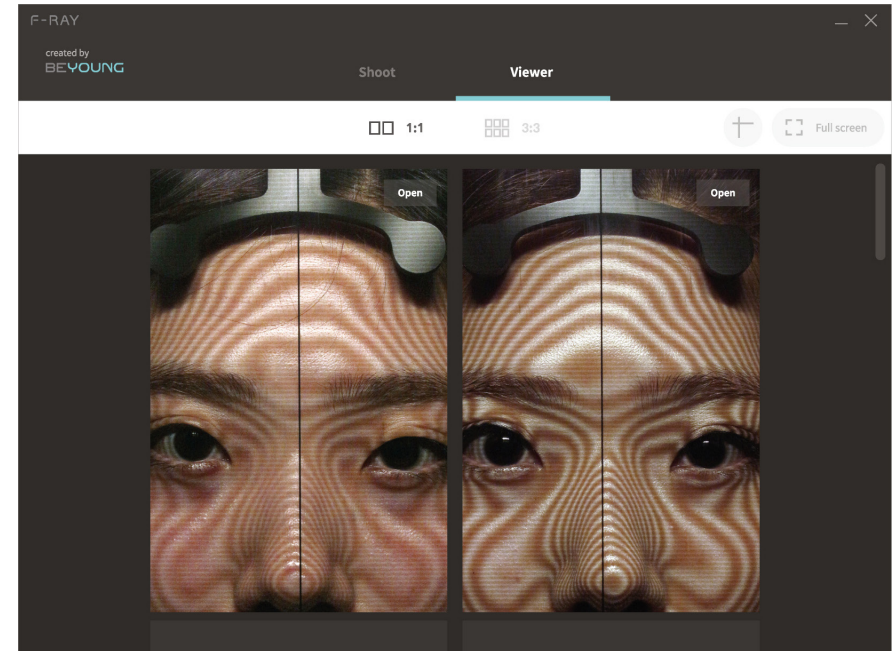Place the cursor over an image and scroll your mouse wheel up to zoom into images. (up to 400%)

- * You can pan a zoomed image by dragging it.
- * When you pan an image, every other image that is loaded will be panned to same position. Use this feature for comparing before/after images.
- * Place the cursor over an image and scroll your mouse wheel down to zoom out from images.

Troubleshooting

- ❗ “F-RAY Connection Failed” error message appears.

Make sure F-RAY is turned on and connected to PC.

- ❗ “Camera Connection Failed” error message appears.

Turn F-RAY off and on again, then retry. If the problem persists, please contact BEYOUNG customer services(+82 2 965 9480).

- ❗ Odd noise occurs when using F-RAY.

Remove F-RAY from power supply immediately and contact BEYOUNG customer services(+82 2 965 9480).

Care and Maintenance

- ❗ Cleaning the device surface

Clean the device surface with a soft cloth slightly moistened with mild detergent, then wipe the surface with a dry cloth. Do not use chemical products (e.g. benzene, thinner, alcohol, etc.) as they may damage the surface or the components.

- ❗ Servicing the device

Do not attempt to service or adjust any electronic or mechanical parts of the device. Should you need the service, please contact BEYOUNG customer services (+82 2 965 9480) for warranty details.

- ❗ Warranty information

The warranty of F-RAY only applies if the product is used in the manner specified in this manual. For detailed terms of warranty, please refer to the warranty card supplied together with the product.

Technical Specifications

Hardware Specs.

Dimensions	350 × 600 × 520 mm
Weight	Approx. 10 kg
Power / Voltage	DC-12V, 220V
Interface	USB; HDMI 1.4
Material	ABS

Camera Specs.

Lens	Canon EF
Camera format	Full frame (1.0x crop factor)
Highest resolution	22.1 MP; 5760 × 3840
Sensor type / size	CMOS, 36 × 24 mm
Image processor	DIGIC 5+

System Requirements

PC

OS	Windows 7 or later (Microsoft .NET Framework 4.5 required for Windows 7)
CPU	Core 2 Duo 2.0 GHz or higher (Core i series or later recommended)
RAM	2.0GB (4.0 GB or more recommended)
Hard-disk space	300MB or more of available hard-disk space for installation

Display

Resolution	XGA (1,024 × 720) display (1,600 × 1,200 or higher recommended)
Colour	16-bit(65,536) colour or higher

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BEYOUNG Co., Ltd.